

# NovoFine® Disposable Needles: A FINE Choice for All Patient Types

## Recommendations for Use

Patient type

Recommended  
needle/technique

Alternative  
needle/technique

Needle/technique  
rationale

Physician  
recommendation

Check one

Slim adult



Slim/normal-weight child

NovoFine® 31 (6 mm)



NovoFine® needles have been shown to result in minimal leakage, if any (less than 1% of the total dose), at the injection site in pediatric and adolescent patients<sup>1</sup>



Normal-weight adult



Overweight adolescent  
(BMI >90th percentile)

NovoFine® 31 (6 mm)



NovoFine® 30 (8 mm)



NovoFine® 31G needles have been shown to result in 100% of injections deposited fully within the subcutaneous layer of both the abdomen and thigh in normal-weight adults<sup>2</sup>




Obese adult  
(BMI >27 kg/m<sup>2</sup>)



NovoFine® 30 (8 mm)




NovoFine® 31 (6 mm)  
Thigh with skinfold

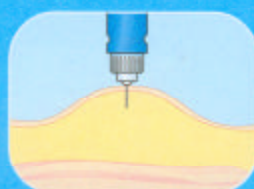

NovoFine® 31G needles have been shown to result in 100% of injections deposited fully within the subcutaneous layer of the thigh with a lifted skinfold and at the abdomen without a skinfold in obese adults<sup>2</sup>



NovoFine® 31 (6 mm)  
Abdomen without skinfold

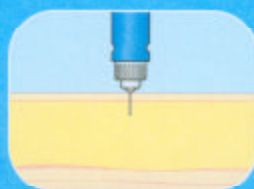

**NovoFine®**  
DISPOSABLE NEEDLE